

Double combination - upholstered enclosures

Modelled on a domestic appearance in a fabric and style of your choice.

Theraposture's stylish range of upholstered beds are designed to look like divan beds found typically in any home. They are built using the same high quality metal mechanisms as we use in the solid oak range – the only difference is the way the beds look.

For those who prefer to have a fabric divan style surround to their bed, there is a choice of: **Belgian Damask, Faux leather, Ultra leather and Waterproof.**

Separable halves

With an upholstered surround it is possible to have two individual matching enclosures that latch together (because the enclosures are much lighter than the oak). This allows for separation of the two sides to gain access to both sides of either half. For situations where carers are involved, this is a great help in reducing the risk of back injury. One or both beds will be mounted on locking castors allowing easy separation.



If you require your bed to be separable, your double headboard will have to be mounted on the wall. The Theraposture latching device provides an easy way to split the two bed halves and then re-secure. The use of castors can provide access for a hoist if needed.

For complimentary furniture, Theraposture's collection of oak bedside cabinets, chests of drawers and wardrobes perfectly suit this upholstered bed range (see our accessories page).



Profiling only

The profiling only action will allow you to independently raise and lower both the back rest and the leg section by simply pushing a button on the hand controller. This mechanism is often the choice of partners in a double combination bed as it allows both individuals to sit up independently together. The metal construction makes this mechanism more durable for long term use and the only choice if grab handles or side rails are needed.

Freephone: 0800 834654

Email: info@theraposture.co.uk www.theraposture.co.uk



Variable height and profiling

In addition to the profiling only mechanism the variable height function is of significant benefit to less able people in a number of ways. If a carer or family member assists a person in bed, the variable height function will smoothly and safely raise the occupant to a safe working height, thereby reducing the risk of a back care injury.

Additionally, this function will help a person to achieve a standing transfer by vertically raising them to standing from sitting in a controlled and safe movement.



Rotoflex turning system

In addition to the variable height and profiling action, the remarkable Rotoflex turning system is a unique solution to an everyday problem – getting legs into and out of bed. This is achieved by a unique rotational action that will sequentially transfer you from a sitting to a sleeping position with the touch of just one button.

In many cases, the Rotoflex truly delivers independence as it is a real alternative to hoisting and carer assistance. In this situation the bed can pay for itself many times over as the cost of care in your home or in a residential home is so high.

The Rotoflex can be made to rotate to either side of the bed to ensure that it is compatible with your room situation.

Options – double combination upholstered adjustable bed

Electronic profiling	Yes	Separable halves	Yes
Electronic variable height	Yes	Single bed	Yes
Rotoflex	Yes	Compatible with cot sides	Yes
2'6" wide	Yes	Compatible with grab handles	Yes
3' wide	Yes	Suitable for castors	Yes
3'6" wide	Yes	Suitable for static feet	Yes
4' wide	Yes	Standard length 6'6"	Yes
4'6" wide	Yes	Shorter length available	Yes
5' wide	Yes	Longer length available	Yes
Double combination	Yes		



E&OE

Freephone: 0800 834654

Email: info@theraposture.co.uk www.theraposture.co.uk